

As we all know all patient's are individual and therefore have their own individual requirements when it comes to their dental laboratory needs. For your convenience and to ensure the quality of work that you have come to expect from IDLabs continues, we have put together an information sheet to help you understand our dental technician's standard protocol and specifications.

All this information can also be found on the back of our IDLab labslip for your easy reference. You will need to remember when filling out our labslip that you indicate your patients unique requirements.

### Occlusal Clearance

The standard for Occlusal clearance is 5 layers Occlusal (Out of Occlusal) paper, unless indicated on IDLabs labslip. Other options are:

- 4 paper (Light Occlusal)
- 3 paper (Tight Occlusal)

### Die Spacer

The standard for Die Spacer is 2 layers, unless indicated on IDLab labslip.

Other Options are:

- 1 Layers
- 3 Layers

### Staining of Occlusal Surface

The standard is Light Staining, unless indicated on IDLabs labslip.

Other options are:

- No Staining
- Medium Staining
- Heavy Staining

### Contacts

The standard Contact is Medium unless indicated on IDLab labslip.

Other Options are:

- Light
- Heavy

### Embrasure

Standard is closed for Anteriors and Open for Posterior unless indicated on IDLab labslip.

Other Options are:

- Open Posteriors
- Closed Posteriors
- Open Anteriors
- Closed Anteriors

### Pontic Design

The standard is a modified ridge.

For more information on sending work to Integrated Dental Laboratories